



Remove all the components from the top of the box **without disassembling them**.




On the bottom of the box you will find the necessary items to collect your sample.




Read carefully the instructions included in the box



1 Place the provided receptor paper on the toilet following the instructions. Make your deposition on it.

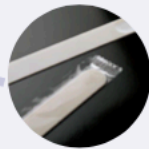

2 Collect part of the stool sample with the spatula.

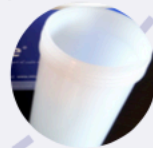

3 Place the stool sample in the container bottle. Do not fill more than half of the container.



4 Place the tray on top of the container bottle.



5 Add the water to the tray to soak the envelope present inside.



6 Place the lid and close tightly.



Your sample is ready to be shipped.